TABLE OF CONTENTS

English...	1	German...	5
French...	2	Product Codes...	6
Spanish...	3	Glossary of Symbols...	6
Italian...	4		

INTENDED USE

CELL-BOND® Slides are used to improve the adhesion of specimens to the slide during routine staining procedures.

SUMMARY

There are various ways to adhere specimens onto glass slides to withstand the rigors of staining. For fecal specimens, a parasitology/fecal fixative (containing an adhesive) may be used on the slide to adhere the specimen to the slide. For AFB concentrated specimens bovine serum albumin, Fraction V has been used to provide a protein adhesive on the slide. These adhesives provide bonding to the slide, but also create a stained protein or chemical background when reviewing the microscopic smear.

The CELL-BOND Slides are ready-to-use, clear slides coated with a unique bio-adhesive that is non-charged but highly wettable. They provide extremely clear backgrounds for routine staining, for immunofluorescent (IFA - DFA) staining or FISH assays, and for histology and cytology use. With no interfering background, the CELL-BOND Slides provide easier viewing and better photographic documentation.

The CELL-BOND Slides eliminate the requirement for protein-based adhesives (i.e., Mayer's Albumin or Slide Coating Solution) for parasitology stained smears fixed in non-PVA fixatives, and eliminate the requirement for bovine serum albumin, Fraction V for AFB smears. CELL-BOND Slides also work effectively to attach bacteria to the slide for critical specimen processing (CSF, synovial fluid, etc.).

FOR IN VITRO DIAGNOSTIC USE ONLY**STABILITY AND STORAGE**

CELL-BOND Slides are stable for 36 months from the date of manufacture. Store at room temperature (15-30°C).

USER QUALITY CONTROL

Each slide should be clean and free of particulate matter.

PROCEDURE

Materials Provided: CELL-BOND Slides

Materials Not Provided: Coverslips, microscope, staining components

SPECIMEN PROCESSING FOR FECAL SMEARS

NOTE: The CELL-BOND Slides are ready for use. No additional treatment is required.

1. Place a drop of the fixed fecal specimen onto the CELL-BOND Slide. Use an applicator stick and "chop" the specimen, spreading it to create thick and thin areas.
2. Air-dry for at least 10-15 minutes.
3. Fix and stain the CELL-BOND Slide according to the procedure described by the manufacturer of the stain being used.
4. Coverslip.

SPECIMEN PROCESSING FOR AFB SMEARS

NOTE: The CELL-BOND Slides are ready for use. No Bovine Serum Albumin, Fraction V is required.

1. Make a thin smear of the specimen onto the CELL-BOND Slide over an area of 2-3 sq. cm.
2. Heat fix by warming on a slide warmer (65-75°C) for 10-15 minutes.
3. Stain the smear on the CELL-BOND Slide according to the procedure described by the manufacturer of the stain being used.

SPECIMEN PROCESSING FOR OTHER BODILY FLUIDS

NOTE: The CELL-BOND Slides are ready for use. No additional treatment is required.

1. Place a drop of the concentrated specimen onto the CELL-BOND Slide and smear over an area of 2-3 sq. cm.
2. Heat fix specimen at 37°C until liquid is dried down.
3. Stain the smear on the CELL-BOND Slide according to the procedure described by the manufacturer of the stain being used.

CONTACT

Alpha-Tec Systems, Inc. offers a complete line of reagents, stains, and QC1™ Quality Control Slides for AFB, Parasitology, Bacteriology, and Mycology processing, as well as O&P collection systems and concentration devices for Parasitology. For Technical Assistance, email Technical@AlphaTecSystems.com, and for Customer Service, email Sales@AlphaTecSystems.com, or call either [+1] 800.221.6058 (USA) or [+1] 360.260.2779 between 8AM and 4PM Monday through Friday, Pacific Time.

WARRANTY

This product is warranted by Alpha-Tec Systems, Inc. to perform as described in the labeling and literature supplied. Alpha-Tec Systems, Inc. disclaims any implied warranty or merchantability or fitness for any other purpose, and in no event shall Alpha-Tec Systems, Inc. be liable for any consequential damages arising out of aforesaid express warranty.

TRADEMARKS

CELL-BOND® and QC1™ are trademarks of Alpha-Tec Systems, Inc., 1311 SE Cardinal Court, Suite 170, Vancouver, WA 98683 USA.

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Manufactured by Alpha-Tec Systems, Inc.
MD59 GmbH