**TABLE OF CONTENTS**

English...	1	Product Codes...	3
French...	2	Glossary of Symbols...	3

**INTENDED USE**

QC1™ Malaria Slides are designed for the quality control of blood parasite staining procedures. They are intended to validate both the quality of the staining reagents and procedure technique.

**SUMMARY**

The microscopic identification of blood parasites is based upon observing the morphology of stained organisms. Staining facilitates identification by differentiating internal structures and separating organisms from background material and artifacts. A number of simple staining techniques have been developed which are satisfactory for diagnostic work.

QC1 Malaria Slides contain parasite-positive substrates necessary to produce the correct staining results when performing the appropriate staining techniques. QC1 Malaria Slides contain a methanol-fixed smear of *Plasmodium falciparum* in various life-cycle stages.

**FOR IN VITRO DIAGNOSTIC USE ONLY****PRECAUTIONS**

QC1 Malaria Slides contain substrates that are applied as a smear. Handle the slides by the edges and do not touch the surface of the slide as this may damage the substrates.

**STABILITY AND STORAGE**

Store slides at room temperature (15-30°C). The slides are stable to the stated expiration date when stored at the required temperature.

**PROCEDURE**

**Materials Provided:** QC1 Malaria Slides.

**Materials Not Provided:** Staining reagents, coverslips, mounting medium, microscope.

**Method of Staining:** QC1 Malaria Slides are pre-fixed and should not be fixed by heat or alcohols prior to staining. Add QC1 Malaria Slides to the first step of the staining procedure. Stain the QC1 Malaria Slides according to the procedure described by the manufacturer of the stain or consult appropriate references.

**RESULTS**

With a traditional Giemsa stain, *Plasmodium falciparum* will stain blue within pinkish-grey red blood cells. Various life-cycle stages of the parasite may be present. Refer to the instruction sheet provided with the stain of choice or consult appropriate references for staining results.

**LIMITATIONS OF PROCEDURES**

Refer to the instruction sheet provided with the stain of choice or consult appropriate references for staining limitations.

**CONTACT**

Alpha-Tec Systems, Inc. offers a complete line of reagents, stains, and QC1™ Quality Control Slides for AFB, Parasitology, Bacteriology, and Mycology processing, as well as O&P collection systems and concentration devices for Parasitology. For Technical Assistance, email [Technical@AlphaTecSystems.com](mailto:Technical@AlphaTecSystems.com), and for Customer Service, email [Sales@AlphaTecSystems.com](mailto:Sales@AlphaTecSystems.com), or call either [+1] 800.221.6058 (USA) or [+1] 360.260.2779 between 8AM and 4PM Monday through Friday, Pacific Time.

**WARRANTY**

This product is warranted by Alpha-Tec Systems, Inc. to perform as described in the labeling and literature supplied. Alpha-Tec Systems, Inc. disclaims any implied warranty or merchantability or fitness for any

other purpose, and in no event shall Alpha-Tec Systems, Inc. be liable for any consequential damages arising out of aforesaid express warranty.

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QC1™ is a trademark of Alpha-Tec Systems, Inc., 1311 SE Cardinal Court, Suite 170, Vancouver, WA 98683 USA.
QC1™ est une marque d'Alpha-Tec Systems, Inc. 1311 SE Cardinal Court, Suite 170, Vancouver, WA 98683 USA.
Manufactured by Alpha-Tec Systems, Inc.
MDSS GmbH