**TABLE OF CONTENTS**

English...	1	Product Codes...	3
French...	2	Glossary of Symbols...	3

**INTENDED USE**

QC1™ Trichomonas Slides are designed for the quality control of *Trichomonas vaginalis* staining procedures. They are intended to validate both the quality of the staining reagents and procedure technique.

**SUMMARY**

The microscopic identification of protozoa is based upon observing the morphology of stained organisms. Staining facilitates identification by differentiating internal structures and separating organisms from background material and artifacts. A number of simple staining techniques have been developed which are satisfactory for diagnostic work.

QC1 Trichomonas Slides contain parasite-positive substrates necessary to produce the correct staining results when performing the appropriate staining techniques. QC1 Trichomonas Slides contain a methanol-fixed smear of *Trichomonas vaginalis* trophozoites.

**FOR IN VITRO DIAGNOSTIC USE ONLY****PRECAUTIONS**

QC1 Trichomonas Slides contain substrates that are applied as a smear. Handle the slides by the edges and do not touch the surface of the slide as this may damage the substrates.

**STABILITY AND STORAGE**

Store slides at room temperature (15-30°C). The slides are stable to the stated expiration date when stored at the required temperature.

**PROCEDURE**

**Materials Provided:** QC1 Trichomonas Slides.

**Materials Not Provided:** Staining reagents, coverslips, mounting medium, microscope/fluorescent microscope.

**Method of Staining:** QC1 Trichomonas Slides are pre-fixed and should not be fixed by heat or alcohols prior to staining. Add QC1 Trichomonas Slides to the first step of the staining procedure. Stain the QC1 Trichomonas Slides according to the procedure described by the manufacturer of the stain, or consult appropriate references.

**RESULTS**

*Trichomonas vaginalis* are pear shaped trophozoites with a single elongated nucleus and 5 flagella. Refer to the instruction sheet provided with the stain of choice or consult appropriate references for staining results.

**LIMITATIONS OF PROCEDURES**

Refer to the instruction sheet provided with the stain of choice or consult appropriate references for staining limitations.

**CONTACT**

CalibreScientific US, Inc. offers a complete line of reagents, stains, and QC1™ Quality Control Slides for AFB, Parasitology, Bacteriology, and Mycology processing, as well as O&P collection systems and concentration devices for Parasitology. For Technical Assistance, email [Technical@AlphaTecSystems.com](mailto:Technical@AlphaTecSystems.com), and for Customer Service, email [Sales@AlphaTecSystems.com](mailto:Sales@AlphaTecSystems.com), or call either [+1] 800.221.6058 (USA) or [+1] 360.260.2779 between 8AM and 4PM Monday through Friday, Pacific Time.

**WARRANTY**

This product is warranted by CalibreScientific US, Inc. to perform as described in the labeling and literature supplied. CalibreScientific US, Inc. disclaims any implied warranty or merchantability or fitness for any other purpose, and in no event shall CalibreScientific US, Inc. be liable for any consequential damages arising out of aforesaid express warranty.

**TRADEMARKS**

QC1™ is a trademark of CalibreScientific AMER IP LLC., 6201 Trust Dr, Holland, OH 43528.

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QC1™ est une marque d' CalibreScientific AMER IP LLC., 6201 Trust Dr, Holland, OH 43528.
Manufactured by CalibreScientific US, Inc.
MDSS GmbH