**Bovine Albumin
Fraction V Solution, 0.2%****TABLE OF CONTENTS**

English...	1	Product Codes...	3
French...	2	Glossary of Symbols...	3

INTENDED USE

Bovine Albumin Fraction V Solution, 0.2% is used in the digestion and decontamination procedure of clinical specimens for the recovery of *Mycobacterium* spp.

PRINCIPLE

A decontamination and digestion procedure utilizing the mucolytic compound N-acetyl-L-cysteine (NALC) per Kubica and Associates results in increased yields of tubercle bacilli. The NALC procedure utilizes N-acetyl-L-cysteine as a mucolytic compound by disrupting chemical bonds in mucus. The sodium hydroxide acts as a bacterial decontaminant and the sodium citrate stabilizes the NALC by chelating (binding) any heavy metal ions present in the specimen. A buffer is used to neutralize the NALC reagents following the appropriate digestion/decontamination time. Bovine Albumin Fraction V Solution, 0.2% allows for the increased isolation of concentrated cells that occurs in the decontamination process and enhances the growth of acid-fast bacteria (AFB).

FOR IN VITRO DIAGNOSTIC USE ONLY**STABILITY AND STORAGE**

Bovine Albumin Fraction V Solution, 0.2% is stable to the stated expiration date when stored at the required temperature. Store unopened bottles at 2-8°C. Refrigerate open bottles at 2-8°C. Do not freeze or overheat. Allow the product to come to room temperature prior to use.

USER QUALITY CONTROL

This product should not be used beyond the expiration date. Any product showing excessive cloudiness or unusual turbidity or coloration should be discarded. Quality controlled microorganisms should be utilized to verify procedures, media, and reagents as appropriate for your laboratory's applicable regulatory agency or local procedural guidelines.

SPECIMEN COLLECTION AND PREPARATION

Appropriate specimens for the detection of *Mycobacterium* spp. should be collected according to prescribed standards and delivered to the laboratory in a safe and timely manner. Refer to local procedural guidelines for this information.

PROCEDURE

Materials Provided: Bovine Albumin Fraction V Solution, 0.2% (10 ml tubes).

Materials Not Provided: Centrifuge, vortex mixer, pipettes, microscope slides, AFB digestion and decontamination reagents, TB media, centrifuge tubes.

SPECIMEN PROCESSING

Process AFB specimens using the NALC-NaOH digestion method or by using the Alpha-Tec Systems, Inc. NAC-PAC® AFB Decontamination System Reagents.

After centrifugation, decant the supernatant and, using a separate sterile pipette for each tube, add 1-2 ml of Bovine Albumin Fraction V Solution, 0.2%. Resuspend the sediment by gentle aspiration with the pipette. Inoculate solid media and make smears for AFB staining.

EXPECTED RESULTS

Bovine Albumin Fraction V Solution can neutralize the toxicity of fatty acids to enhance growth, and aid in cell adhesion on microscope slides.

LIMITATIONS OF PROCEDURES

Timing of the decontamination step, proper buffering, speed and timing of the centrifugation step, proper decanting and the addition of an appropriate resuspension solution are vital to the recovery of *Mycobacterium* spp. Failure to follow appropriate decontamination procedures may result in decreased numbers or total loss of *Mycobacterium* spp. resulting in an inaccurate culture report.

SPECIFIC PERFORMANCE CHARACTERISTICS

This product was tested on clinical samples and recovered all culture-appropriate *Mycobacterium* spp. when the designated procedures were followed.

BIBLIOGRAPHY

1. Kent, P., Kubica, G.P., Public Health Mycobacteriology: A Guide for the Level III Laboratory. Centers for Disease Control and Prevention (CDC), 1985.
2. Kubica, G.P., et al. "Sputum Digestion and Decontamination with N-acetyl-L-cysteine-Sodium Hydroxide for Culture of Mycobacteria." Am Rev Respir Dis. 87:775-779, 1963.
3. Kubica, G.P., et al. "Comments on the Use of the New Mucolytic Agent, N-acetyl-L-cysteine, as a Sputum Digestant for the Isolation of Mycobacteria." Am Rev Respir Dis. 89:284-286, 1964.
4. Davis and Dubos. J. Bacteriol. 55:11, 1945
5. Murray, P., Manual of Clinical Microbiology. American Society for Microbiology, 1985.
6. Mycobacterium Tuberculosis: Assessing Your Laboratory. Centers for Disease Control and Prevention (CDC), March, 1995.

CONTACT

Alpha-Tec Systems, Inc. offers a complete line of reagents, stains, and QC1™ Quality Control Slides for AFB, Parasitology, Bacteriology, and Mycology processing, as well as O&P collection systems and concentration devices for Parasitology. For Technical Assistance, email Technical@AlphaTecSystems.com, and for Customer Service, email Sales@AlphaTecSystems.com, or call either [+1] 800.221.6058 (USA) or [+1] 360.260.2779 between 8AM and 4PM Monday through Friday, Pacific Time.

WARRANTY

This product is warranted by Alpha-Tec Systems, Inc. to perform as described in the labeling and literature supplied. Alpha-Tec Systems, Inc. disclaims any implied warranty or merchantability or fitness for any other purpose, and in no event shall Alpha-Tec Systems, Inc. be liable for any consequential damages arising out of aforesaid express warranty.

TRADEMARKS

NAC-PAC® and QC1™ are trademarks of Alpha-Tec Systems, Inc., 1311 SE Cardinal Court, Suite 170, Vancouver, WA 98683 USA.

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Manufactured by Alpha-Tec Systems, Inc.
MDSS GmbH