Directions

for Stool Specimen Collection Set

1

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3

Pass the stool specimen into any clean and dry container such as a bedpan, a plastic bag, a plastic plate or a newspaper.

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4

Carefully open each vial. Using the spoon attached to the cap, collect small amounts of stool from areas that are slimy, watery or bloody and place them into each vial. If the stool is hard, collect small amounts from both ends and the middle and place into each vial.

IMPORTANT: Fill each vial with enough specimen until the liquid reaches the *Fill Line* located on the label.

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
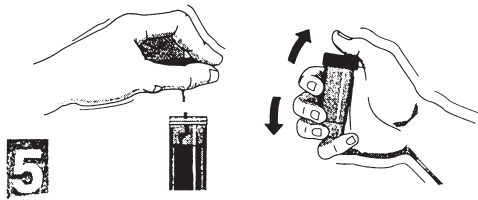
5

Fill only one vial at a time and replace the spoon/cap onto the same vial that it came from. Do not mix spoons/caps with other vials.

After replacing the cap onto the vial, tighten it to avoid any spilling.

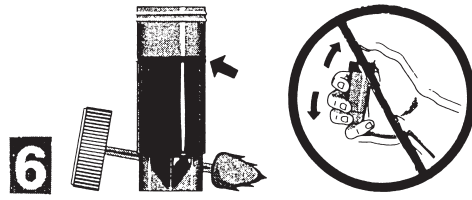
IMPORTANT: Shake the sample thoroughly until the sample is well-mixed with the liquid in the vial.

6


If one vial contains no liquids (empty vial), fill the vial with stool specimen up to the **Fill Line** on the vial label. **DO NOT** mix or shake this vial. Tightly close the cap on the vial.

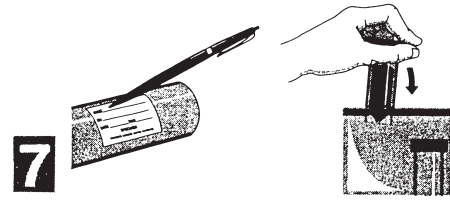IMPORTANT: Make certain that all vials are tightly capped to avoid leakage.

7

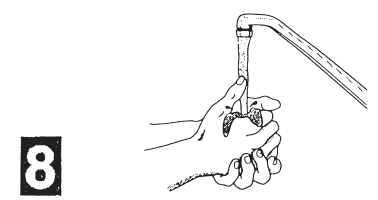Fill out each label with all information requested. Check the box that best describes your specimen. Place all of the vials back into the package box or bag and close the lid. Wash your hands well with soap and water. Return the vials to your physician or laboratory where you received them.

8

IMPORTANT: If any liquid from the vials or stool specimen gets into your eyes or on your skin, rinse the area with plenty of water. If any discomfort or irritation develops, contact a physician.

ANTIDOTE IF SWALLOWED: Give a glass of milk. Follow with a tablespoon of salt in a glass of warm water. Repeat until the vomit is clear. Contact a physician or local poison center IMMEDIATELY.

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